


TECHNICAL BULLETIN

USING ACRYLIC WITH HEX-LOCK PLASTIC TEMPORARY ABUTMENTS

When preparing an acrylic restoration on the Hex-Lock Plastic Temporary Abutment, the following steps are required:

- After trimming the abutment, roughen the surface to receive the acrylic.
- Block out the areas indicated below with soft utility wax to prevent ingress of the acrylic material.
- Prepare the acrylic mixture (monomer and polymer) in a separate container prior to application. The monomer (liquid) should not be applied directly to the surface of the abutment to “wet” the surface.

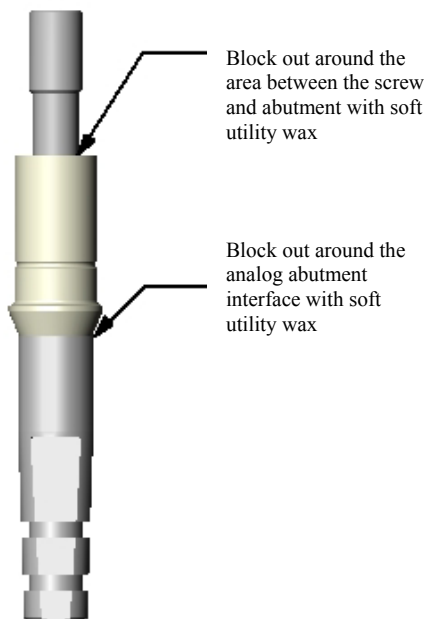

For complete instructions, please refer to the Instructions For Use, Part Number 4885 or Part Number 4894.